



aspire

MOBILE SHOWER COMMODES

USER MANUAL





Thank you for choosing this Stainless Steel Commode from our Aspire Mobile Shower Commode Range.

This product has been specifically designed to offer comfort, support and safety.

TABLE OF CONTENTS

1. Introduction	5
2. Safety Precautions	6
3. Product Overview	8
4. Unpacking & Inspection	11
5. Assembly Instructions	15
6. Operation	19
7. Accessories	21
8. Troubleshooting	22
9. Cleaning	23
10. Service & Maintenance	24
11. Specifications	25
12. Warranty	26

1. INTRODUCTION

The Aspire Mobile Shower Commode range is designed for a healthcare environment such as a facility or hospital. The commodes have a fabricated stainless steel frame and utilise plastic and PU components of various material specifications.

Clinical Applications and Area of Use

These commodes are appropriate for adult patient care and should only be used under the instruction of a carer or health professional.

Key Features

- Maximum Safe Working Load (SWL) of 200kg, 250kg & 400kg (dependent on model)
- Attendant propelled and self propelled options available
- Height adjustability to accommodate different users (TIS model only)
- Stainless steel finish
- Swing back safety arms for ease of cleaning and transfer
- Soft PU seat range in open or closed variations (sold separately)
- Easy slide weight bearing platform or swing away legrest options available
- Access trough with improved over toilet clearance for easy hygiene access
- 8° - 18° backrest angle ensures patient comfort
- Angle adjustable backrest (swing away models only) for improved seating comfort
- Ultra low seat height of 590mm or down to 490mm when customised (swing away models only)

2. SAFETY PRECAUTIONS

It is important that this User Manual is read in its entirety prior to using the commode. The manual is designed to ensure that users are equipped with the information required to competently and responsibly operate this commode.

It is recommended that the User Manual be kept in an easily accessible place for quick reference. If required, Aspire product specialists can provide additional training.

All warnings and cautions within this manual should be strictly adhered to. Failure to do so could result in harm to the patient or operator.

Warnings highlighted throughout this manual identify possible hazards in procedures or conditions which if not followed correctly could result in serious injury or fatality.

Cautions highlighted throughout this manual identify possible hazards in procedures or conditions, which if not followed correctly could result in equipment damage or failure.

GENERAL

- This commode is designed to be used indoors
- Suitable for wet areas but not pools
- The clinically qualified person responsible should consider the age, size and condition of the patient before using the commode
- The maximum user weight is 200kg, 250kg or 400kg (dependent on model)


- **ALWAYS** check with your healthcare professional prior to using this commode.
- **ALWAYS** apply the brakes when the commode is stationary.
- **ALWAYS** ensure brakes remain locked when transferring into or out of the commode.
- **ALWAYS** ensure that the legrests are swung away when transferring into and out of the commode.
- **ALWAYS** ensure that attendant and patient limbs are free from potential entrapment areas before operating any functions.
- **DO NOT** stand on the product.
- **DO NOT** use outdoors.
- **DO NOT** use this product without reading the user manual in its entirety.
- **DO NOT** use if defective.
- **DO NOT** make any changes or modifications to this product without consultation from your Aspire Dealer.

3. PRODUCT OVERVIEW


The Aspire Mobile Shower Commode range is designed specifically for the Aged Care and Hospital environments. Please take time to familiarise yourself with key components shown below:

SWING AWAY LEGREST MODELS

Self Propelled Wheels (selected models only)

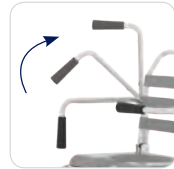

Swing back Safety Arms

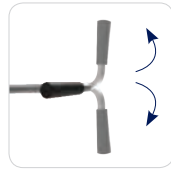

Adjustable Safety Arms

Access trough with improved over toilet clearance for easy hygiene access



Soft PU Seat (Sold Separately)



4 x Stainless Steel Lockable Castors




Swing Away Legrests




Adjustable Height Legrests



WEIGHT BEARING PLATFORM MODELS

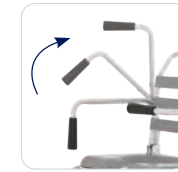

Swing back Safety Arms

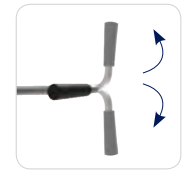


Adjustable Safety Arms

Self Propelled Wheels (selected models only)



Access trough with improved over toilet clearance for easy hygiene access

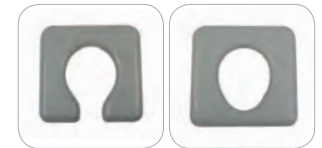

Soft PU Seat (Sold Separately)




4 x Stainless Steel Lockable Castors



Easy-Slide Weight Bearing Platform

TILT & RECLINE MODELS

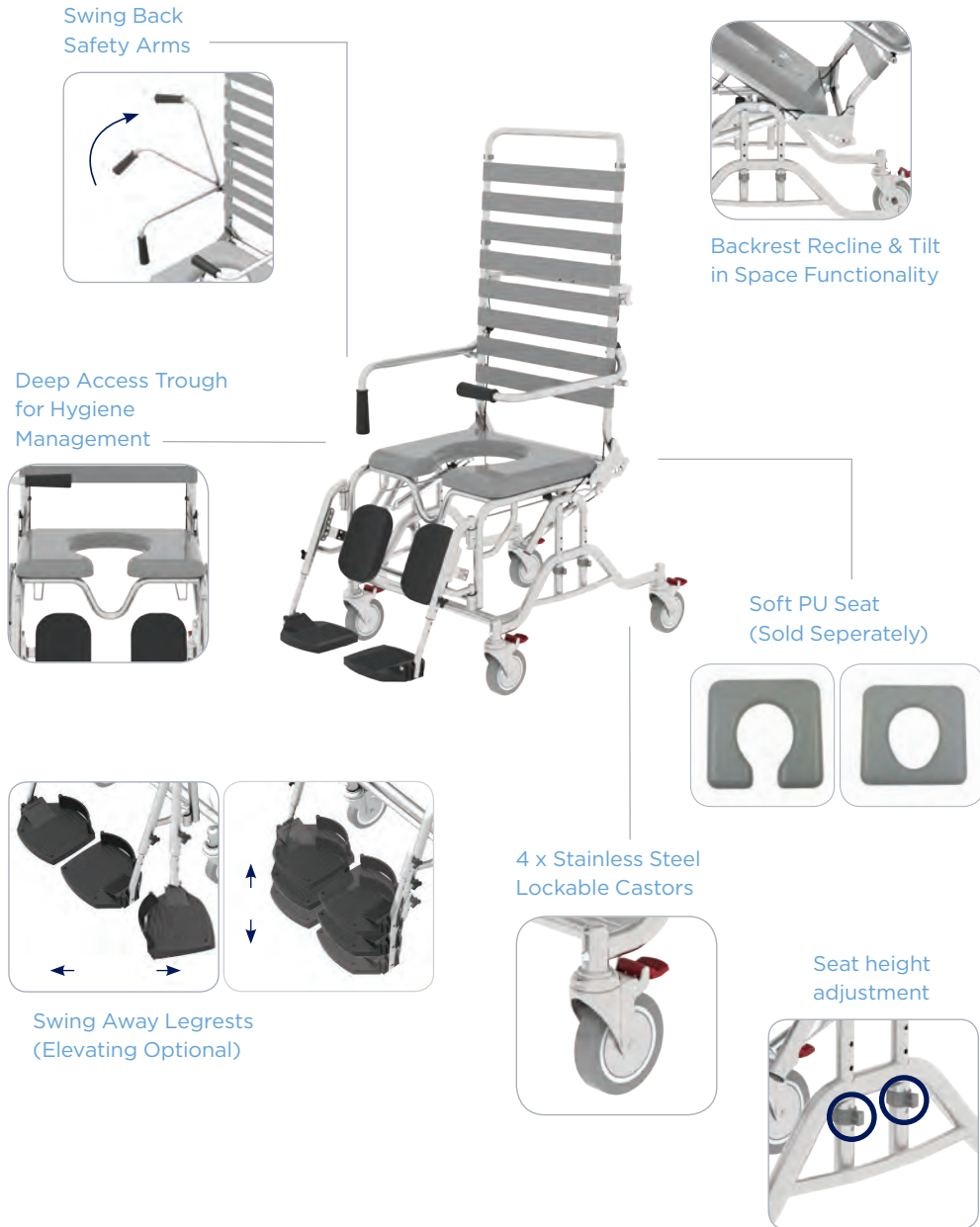

4. UNPACKING & INSPECTION

Aspire Mobile Shower Commodes come packaged in 1 box for maximum protection. Check box to ensure no components are missing and contact Aidacare if this occurs on 1300 133 120.

SWING AWAY LEGREST MODELS

IN CARTON

- 1 Commode frame with backrest, safety arms & locking castors
- 2 Swing away legrests (pair)
- 3 User manual (in plastic bag)
- 4 Seat mounting hardware (in plastic bag)

SOLD SEPARATELY

- 5 Open or closed seat
- 6 Pan carrier
- 7 Commode bowl/bed pan

IN CARTON



SOLD SEPARATELY



WEIGHT BEARING PLATFORM MODELS

IN CARTON

- 1 Commode frame with backrest, safety arms & locking castors
- 2 User manual (in plastic bag)
- 3 Seat mounting hardware (in plastic bag)

SOLD SEPARATELY

- 4 Open or closed seat
- 5 Pan carrier
- 6 Commode bowl/bed pan

IN CARTON



SOLD SEPARATELY



TILT & RECLINE MODELS

IN CARTON

- 1 Commode frame with backrest, safety arms, locking castors
- 2 Swing away legrests (pair)
- 3 User manual (in plastic bag)
- 4 Seat mounting hardware (in plastic bag)


SOLD SEPARATELY

- 5 Open or closed seat
- 6 Pan carrier
- 7 Commode bowl/bed pan

IN CARTON



SOLD SEPARATELY

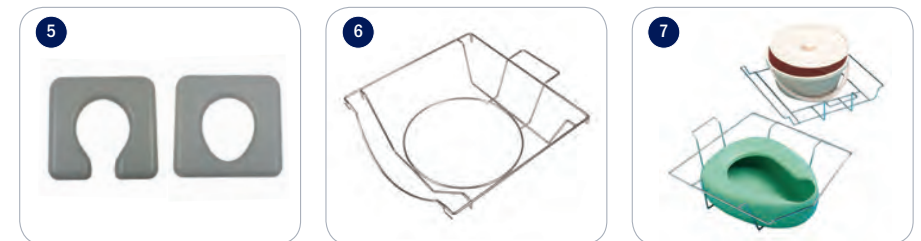

SELF PROPELLING MODELS

IN CARTON

- 1 Commode frame with backrest, safety arms, locking castors
- 2 Swing away legrests (swing away legrest models only) (pair)
- 3 Seat mounting hardware (in plastic bag)
- 4 User manual (in plastic bag)

ASSEMBLY REQUIRED

- 5 Self propelling kit (wheels, brakes & anti-tippers)

SOLD SEPARATELY

- 6 Open or closed seat
- 7 Pan carrier
- 8 Commode bowl/bed pan

IN CARTON



ASSEMBLY REQUIRED



SOLD SEPARATELY



5. ASSEMBLY INSTRUCTIONS

SWING AWAY LEGREST MODELS

Step 1

Remove commode from box & clear away all packaging materials.




Step 2

Detach screws that are fitted into backrest, swing arms and backrest into position and secure in place with screws.



Step 3

Attach legrests into lugs mounted on commode frame 1 & tighten with screws underneath top lug 2

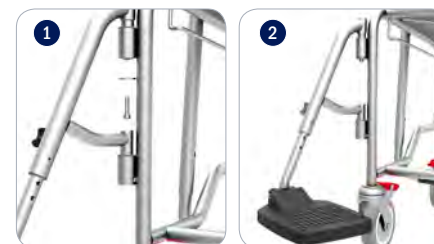

Step 4

Using seat mounting hardware, affix the seat 3 (sold separately) to the top of the frame and screw in place until tight 4



Step 5

Slide in pan carrier & bowl (sold separately).



WEIGHT BEARING PLATFORM MODELS

Step 1

Remove commode from box & clear away all packaging materials.



Step 2

Detach screws that are fitted into backrest, swing arms and backrest into position and secure in place with screws.



Step 3

Using seat mounting hardware, affix the seat 1 (sold separately) to the top of the frame and screw in place until tight 2.



Step 4

Slide in pan carrier & bowl (sold separately).



TILT & RECLINE MODELS



Step 1

Remove commode from box & clear away all packaging materials.



Step 2

Raise backrest into desired position 1, it will click into place when secure. Slide in top frame and secure with C Clips 2.

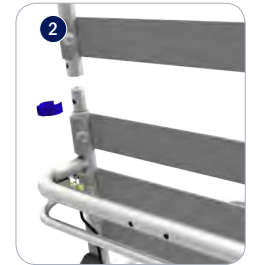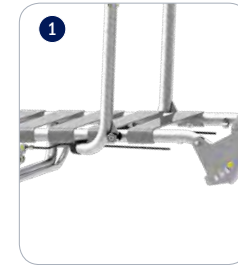

Step 3

Attach legrests into lugs mounted on commode frame 3 & tighten with screws underneath top lug 4.



Step 4

Using seat mounting hardware, affix the seat 5 (sold separately) to the top of the frame and screw in place until tight 6.



Step 5

Slide in pan carrier & bowl (sold separately).



SELF PROPELLING MODELS

Step 1

Remove commode from box & clear away all packaging materials.



Step 3

Attach legrests into lugs mounted on commode frame ① & tighten with screws underneath top lug ② (swing away legrest models only).



Step 5

Slide in pan carrier & bowl (sold separately).



Step 2

Detach screws that are fitted into backrest, swing arms and backrest into position and secure in place with screws.




Step 4

Using seat mounting hardware, affix the seat ③ (sold separately) to the top of the frame and screw in place until tight ④



Step 6

Remove locking pin from frame ⑤, insert through centre of wheel ⑥, mount brake with clamp onto each side of frame ⑦ detach rear castors and replace with anti-tippers ⑧

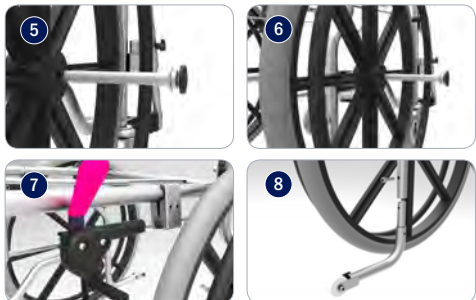

6. OPERATION

Installing the Commode Seat (sold separately)

- Place seat onto top of frame and align along guides ①
- Using tri-wheel mounting kit (supplied with commode) screw the tri-wheels firmly into place until seat is secured onto the frame ②



Activating the Locking Castors - Attendant Propelled Models

- To engage, press the ON lever down with your foot ①
- To disengage, press the OFF lever down with your foot ②



Activating the Brakes - Self Propelled Models

- Push down and forward on the black hand levers on both sides of the commode to lock rear wheels in place ①
- Pull back on the hand lever to disengage brakes ②



Adjusting the Legrests (Swing Away Legrest Models Only)

- Lift and swing the legrests to desired position ①
- If the user wishes to vacate the chair, swing the footrests outwards for clearance. They can be completely removed by removing locking screw & lifting upwards ②



Installing & Removing Pan Carrier & Commode Bowl/Pan (sold separately)

- Holding the handle of pan carrier, insert the front end into the mounting rails at the rear of the commode ①
- Place commode bowl or pan into pan carrier before sliding pan carrier underneath seat ②



Using Safety Armrests

- Swing armrests to desired position and ensure spring pin locks into place ①



7. ACCESSORIES




Open Seat
BTC066600 - 460mm
BTC066750 - 530mm
BTC066800 - 600mm



Closed Seat
BTC066700 - 460mm



Hand Made Pressure Reducing Seat - 460mm
BTC089130 - Open Front
BTC089140 - Closed Front
BTC089150 - Open Left
BTC089160 - Open Right

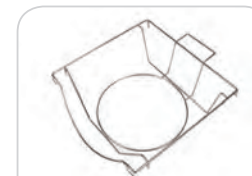


Commode Bowl Carrier
BTC066400 -
460 - 600mm Commodes




Oxygen Bottle Carrier
BTC067200



Armrest - Padded Offset
BTC067000

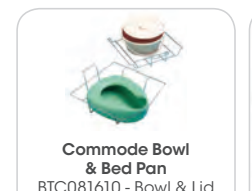

Commode Bowl & Bed Pan
BTC081610 - Bowl & Lid
BTT151800 - Bed Pan

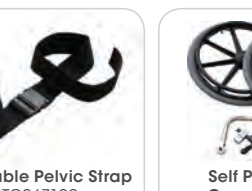

Adjustable Pelvic Strap
BTC067100



Self Propelling Conversion Kit
BTC082900

Elevating Legrests
BTC067110 - Left
BTC067115 - Right



Adjustable Thoracic Support WCA867010



Advanced Headrest
BTC067050



Stump/Footrest Support
BTC067105

Adjustable IV Pole
BTC067103

8. TROUBLESHOOTING

Fault	Cause	Solution
Difficulty in removing/adjusting footrests	Build up of material in receivers	<ul style="list-style-type: none"> Remove footrests, clean receivers and footrest mounts
Difficulty adjusting the seat height	Build up of material in seat frame adjustment point	<ul style="list-style-type: none"> Remove upper from lower frame and clean thoroughly
Difficulty in maneuvering castors	Build up of dirt between fork and castor	<ul style="list-style-type: none"> Clean castors thoroughly
Difficulty in adjusting commode seat	Build up of material between frame and seat	<ul style="list-style-type: none"> Remove seat and clean thoroughly

 Should problems persist contact Aspire Service on 1300 133 120.

9. CLEANING

WARNING

Note: It is strongly recommended that protective clothing be used when cleaning.

- Frame and metal components - Always keep the commode clean for greater comfort and long-lasting performance. Clean stainless steel components with water and a mild detergent. Wipe dry.
- Brakes - The braking effect depends on the state of the tyre tread. Excessive dirt build-up may negatively affect the application of the brakes. If necessary, clean tyres with a small brush to remove dust and waste.
- Seat and backrest padding - Easy clean material that can be washed with water and neutral soap/detergent.
- Commode Bowl - Remove and clean with water and a neutral soap/detergent.

CAUTION

Do not use abrasive Hypocarbonate or Phenolic compounds or pads.

10. SERVICE & MAINTENANCE

To prolong the working life of the commode and for trouble free operation it is recommended that routine inspections should be performed by a person who is suitably qualified and well acquainted with the design, use and care of Aspire Mobile Shower Commodes.


If inspection reveals that the safety of the commode is compromised in any way, it should be IMMEDIATELY WITHDRAWN FROM SERVICE, tagged as such and the owner notified, until repair or replacement has occurred.

Inspection:

A visual and functional inspection should be performed monthly and every time the commode changes user.

- Visually inspect mechanical components: Can you see damage, chips or deformation to the frame, armrests, seat, backrest, legrests or commode bowl?
- Are the castors, wheels and brakes functioning as they should?
- Are all frame securement hand wheels present and tightened?

11. SPECIFICATIONS

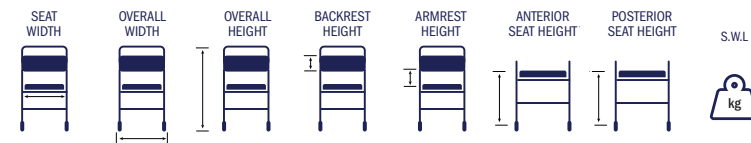

	SEAT WIDTH	OVERALL WIDTH	OVERALL HEIGHT	BACKREST HEIGHT	ARMREST HEIGHT	ANTERIOR SEAT HEIGHT	POSTERIOR SEAT HEIGHT	S.W.L.
BTC066000 Attendant Propelled - WB Platform	460mm	535mm	995mm	305mm	225mm	590mm	560mm	200kg
BTC066200 Self Propelled - WB Platform	460mm	665mm	995mm	305mm	225mm	590mm	560mm	200kg
BTC066050 Attendant Propelled - WB Platform	530mm	605mm	995mm	305mm	225mm	590mm	560mm	200kg
BTC066055 Self Propelled - WB Platform	530mm	735mm	995mm	305mm	225mm	590mm	560mm	200kg
BTC061400 Attendant Propelled - WB Platform	600mm	725mm	995mm	300mm	290mm	590mm	560mm	400kg
BTC065900 Attendant Propelled - Swing Away Footrests	460mm	535mm	895 - 995mm	305mm	225mm	490- 590mm	560mm	200kg
BTC066100 Self Propelled - Swing Away Footrests	460mm	665mm	895 - 995mm	305mm	225mm	490- 590mm	560mm	200kg
BTC061300 Attendant Propelled - Swing Away Footrests	600mm	675mm	995mm	305mm	225mm	490- 590mm	560mm	200kg
BTC061305 Self Propelled - Swing Away Footrests	600mm	805mm	995mm	305mm	225mm	490- 590mm	560mm	200kg
BTC066030 Attendant Propelled - Tilt & Recline	460mm	560mm	1315- 1485mm	900mm	275mm	460- 630mm	460- 620mm	250kg
BTC066040 Attendant Propelled - Tilt & Recline	530mm	630mm	1315- 1485mm	900mm	275mm	460- 630mm	460- 620mm	250kg
BTC066060 Attendant Propelled - Swing Away	530mm	605mm	995mm	305mm	225mm	490- 590mm	560mm	200kg
BTC066065 Self Propelled - Swing Away	530mm	735mm	995mm	305mm	225mm	490- 590mm	560mm	200kg

*Swing away model are only adjustable when customised

12. WARRANTY

Warranty

Aspire Mobile Shower Commodes come with a five year corrosion resistance warranty on the external frame surface. If the product surface becomes damaged this will void the warranty. A general one year warranty also applies covering faulty workmanship or defective materials. This warranty is valid for twelve months from the original purchase date, and to the original customer.

Aidacare is bound by Australian Consumer Law in respect to 'Warranty against Defects' on this product. A copy of the Aidacare 'Warranty against Defects' statement is available at www.aidacare.com.au.

Please contact Aidacare directly for support in regard to warranty matters on 1300 133 120.

Aidacare does not warrant against faults caused by incorrect use, unauthorised modification or any situation that could not be deemed as fair wear and tear.

The warranty on this product is a back to base warranty and does not cover any freight costs pertaining to the return of any items under warranty. Aidacare acknowledges and conforms to all ACCC guidelines around consumer rights. For further information contact Aidacare 1300 133 120.

Aidacare will not guarantee the safety and the correct functioning of the product in the case that any of the original components have been changed or modified and or replaced with non-original Aspire approved parts. If any faults are detected upon receiving the product please phone Aidacare to arrange a replacement. Any faults that are detected during normal use should be reported to Aidacare immediately to determine if warranty conditions apply and organise repair and/or replacement as applicable.

For servicing, preventative maintenance and any other questions regarding this or any other Aidacare product please contact:

Aidacare Pty Ltd
4 Noonan Rd, Ingleburn NSW 2565
T - 1300 133 120
W - www.aidacare.com.au
E - product@aidacare.com.au

Spare parts list available on request.





AIDACARE PTY LTD

4 Noonan Road, Ingleburn NSW 2565

1300 133 120

www.aidacare.com.au

MADE IN PR CHINA