



11/05/2021

ASPIRE TIS SHOWER COMMODE – PRODUCT UPDATE!

Our Aspire TIS Shower Commodes have undergone a significant design upgrade to enhance safety and improve performance and the user experience.

Functionally the commode performs the same actions (tilt-in-space, back recline, seat height adjustment etc) however where previously the base frame and seating frame were joined via dual upright posts, they are now joined via single upright posts (see old/new comparison pics below).

The refined and more open design utilises less materials and component parts and is more accessible for maintenance, while the spacing around moving parts eliminates any hazards for limbs.

KEY DIFFERENCES & IMPROVEMENTS

- o **Safety** – Hazard risks for limbs around moving parts have been eliminated.
- o **Adjustability** – Now operating on a single post, height adjustment is a lot easier to adjust to suit changing client needs, particularly for one carer.
- o **Patient Weight** – SWL has been **reduced from 250kg to 200kg** to facilitate the design change, however 200kg is still greater than or equal to other competing brands. We believe this is a worthwhile compromise for the improved adjustability and safety, versus the likely maximum user mass that can possibly fit into the 460mm or 530mm seat widths.

OLD STYLE – DUAL POST

NEW STYLE – SINGLE POST

